

# TrachStay™

## Sturdy, efficient device stabilizes ventilator connector; stabilizes in-line suction devices

TrachStay provides an easy to use, convenient solution to minimize the potential for inadvertent ventilator circuit disconnects for adult and pediatric patients. TrachStay is a highly efficient and sturdy anti-disconnect device that attaches with all tracheostomy securing devices. TrachStay stabilizes the ventilator connector, in-line suction system or other supplemental gas delivery systems. TrachStay is latex free. Application is easy and secure.

To attach, place the TrachStay ring over the tracheostomy tube until it rests on the flange. Release one side of the tracheostomy collar, slip the hook of the collar through the flange and TrachStay ring, and reattach to the collar. Repeat on the other side of the collar. Attach the ventilator, in-line suction system or other supplemental gas delivery device to the tracheostomy tube. Then place the TrachStay straps over the attached device and secure its hook-and-loop fastener to stabilize the connection.

TrachStay protects against accidental disconnects and cross contamination.

### SAFE

Latex free, hypoallergenic materials

Enhanced durability for securing the ventilator circuit connection

Minimizes potential ventilator attachment separation

Reduces potential exposure to contaminated tubing moisture

### COST EFFECTIVE

Designed for use in critical care, PICU

Designed for use in sub acute, long term acute, rehab and home care

### CONVENIENT

Minimizes time for setup and connection

Easy and quick application to all tracheostomy collars

### VERSATILE

Adult and pediatric application

Can be used with delivery of aerosol and supplemental oxygen therapy

Anti-disconnect device can be used with all tracheostomy collars

### Ordering Information

11270	TrachStay (50/case)
-------	---------------------

### Complementary Products

20001	Sil.Flex Stoma Pad™ – Size 1 (10/box)
-------	---------------------------------------


20002	Sil.Flex Stoma Pad – Size 2 (10/box)
-------	--------------------------------------

20003	Sil.Flex Stoma Pad – Size 3 (10/box)
-------	--------------------------------------

### B&B Products are available from

B&B Medical Technologies and finer specialty care distributors.

Visit [www.BandB-Medical.com](http://www.BandB-Medical.com) or contact us today at 1.800.242.8778.

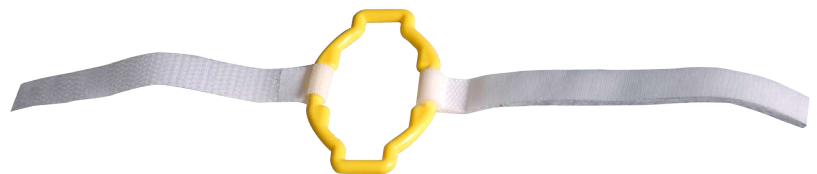

11270C2

© 2009 B&B Medical Technologies. All rights reserved.  
Sil.Flex Stoma Pad is a trademark of A Plus Medical.  
"A World of Products for Better Breathing" is a service mark of B&B Medical Technologies.

Toll-free: +1.800.242.8778  
Tel: +1.760.929.9972  
Fax: +1.760.929.9953  
2734 Loker Avenue West, Suite M  
Carlsbad, CA 92010 USA

**B&B**  
MEDICAL TECHNOLOGIES  
A World of Products for Better Breathing™