



Tracheostomy Tube Holders

Distributed by:

B&B Medical Technologies
2734 Loker Ave West, Suite M
Carlsbad, CA 92010 USA
Tel: +1.760.929.9972
Fax: +1.760.929.9953
www.BandB-Medical.com

Sm 25001

Md 25002

Lrg 25003

Instructions for Use

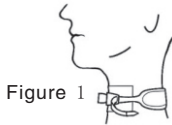


Figure 1

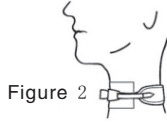


Figure 2

1. Insert the pointed hook end of the tie through the eyelet of the tracheostomy tube. Attach the hooked end to the brush side of the padded neck collar or to the strap (figure 1).
2. Wrap the padded neck collar around the patient's neck and then insert the other pointed hook end of the tie through the other eyelet of the tracheostomy tube. Pull the tie snug and attach it to the brushed side of the padded neck collar or the strap (figure 2).
3. Routinely check and readjust the tie so that one finger can easily fit between the collar and the neck and make sure that the hook ends of the tie are secured to the brushed side of the padded neck collar.

! CAUTION: When changing the tie on a patient without changing the tracheostomy tube, be sure to hold the tube in place while changing the tie to insure that decannulation does not occur.

For Single Patient Use: Danny Ties may be washed if soiled, but must be discarded if the hook end of the tie does not have good adhesion to the brushed side of the padded neck collar.

IMPORTANT NOTICE: Please read the instructions for use carefully. These instructions should be read and understood by all persons who apply this product. Always carefully monitor patients to ensure they are not at risk for disrupting lifesaving treatment. Patients must have constant supervision by trained personnel, electronic monitoring and/or the use of a restraint as well as proper medical supervision.



Manufactured in China by Qingdao Leckie Foam Products Co., Ltd.



SUNGO GROUP CO., LTD. 78 York St. London, W1H 1DP England